


Thank you for choosing INSPECTIS®

This product has been tested with Windows 10/11 Professional. Below you will find step-by-step instructions on how to install.

Installation shall be done with full administration privileges. You might need to disable the antivirus system of your computer temporary during the installation.

1. Plug the provided software protection Dongle into a free USB port of your computer.
2. Run the Setup file. The installation wizard will start and guide you through the installation.
3. Capillaroscope device Drivers are installed automatically by installation wizard. If Drivers are already installed on your computer, uncheck the Capillaroscope Device Driver on the drivers installation dialogue. Notice no Drivers are needed for CapiScope and DigiScope devices.
4. Connect your Capillaroscope to a free USB3.0, USB3.1 or USB3.2 port of your computer. Minimum **5.0 Gbit/s data transmission** is required.
5. Follow on-screen instructions to complete the installation.
6. Click on INSPECTIS  icon on your computer desktop and select **Run as Administrator first time**. Running as Administrator will enable you to register and activate the software automatically over internet.
7. Select language from the list.
8. From the "Software Version" selection window, select:
 - "INSPECTIS CAP Pro" or "INSPECTIS CAP Basics" depending on your software licence.
 - 15 days trial version for trying the software for 15 days. All software functions will be enabled for 15 days. Skip part 5 (license number).
9. Fill in license number (found in the package) and the other requested information.
10. If automatic on-line registrations is not possible, complete the Off-line Registration form and email it to info@inspect-is-com to get an software activation key.
11. Follow on-screen instructions to complete the installation.

For more information please refer to instruction's manual of **INSPECTIS®** software, contact your dealer or send an email to info@inspect-is-com.