

## **Release Notes for INSPECTIS© CAP Pro, V7.2.6 through V7.2.8**

### **New:**

1. Camera device drivers are installed automatically by wizard.
2. 1600x1200 format is added to "Device Resolution" of the camera.
3. A measurement file in CSV format is created automatically and saved in real time. The location of the measurement file is defined by the user under Settings.

### **Improvements:**

4. Destination of the Log file is changed to AppData\Roaming.
5. Warning tab pops up if all fields in patient data are not filled.
6. Patient Name, ID, Date of Birth and Examination Date are saved together with the patient examination data when exported to Excel workbook.

### **Bugfixes:**

7. Copying database caused program crash sometimes.
8. Help indexing issues resolved.
9. Sporadic software crashes due to double click on image thumbnail resolved.
10. Sporadic software crashes when database is copied is resolved.
11. Adding additional fields in Capillary Preferences was not displayed in the Add Patient dialog tab.